

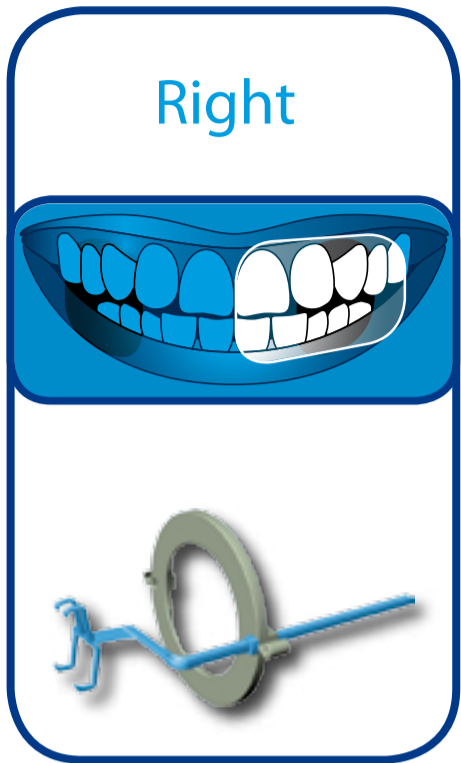




Bitewing

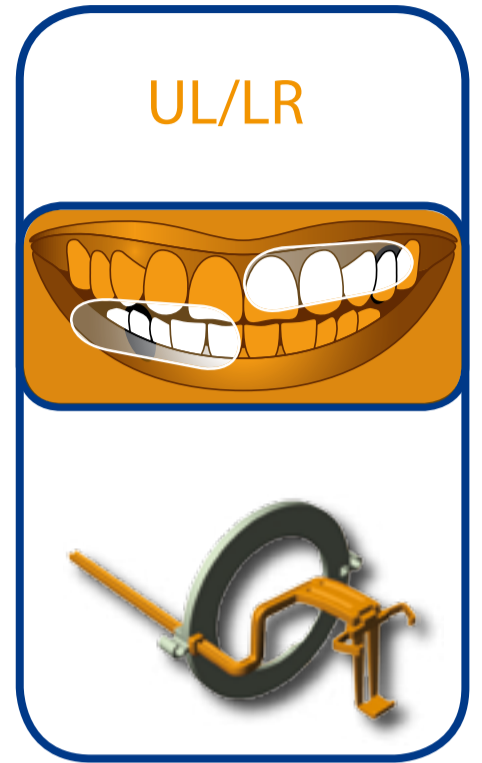


Periapical

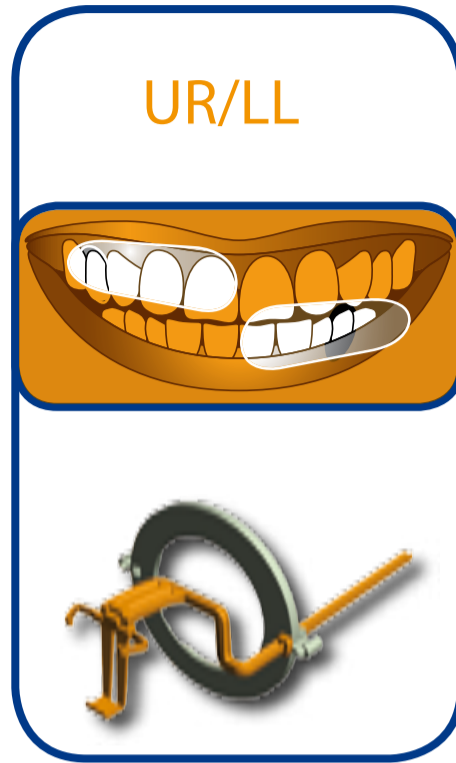


Posterior

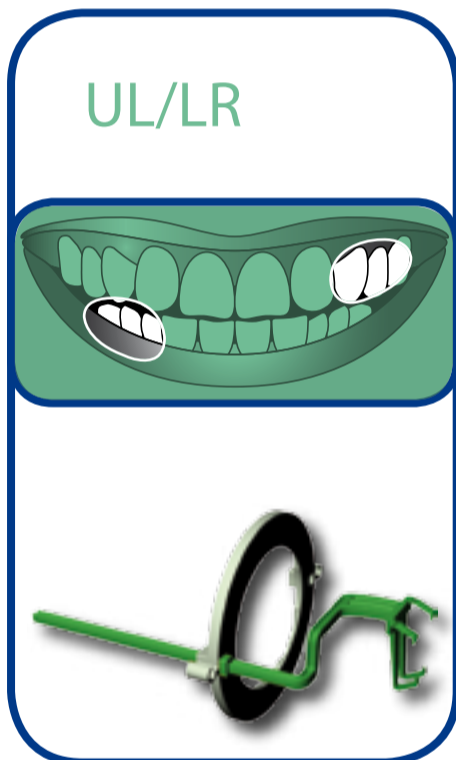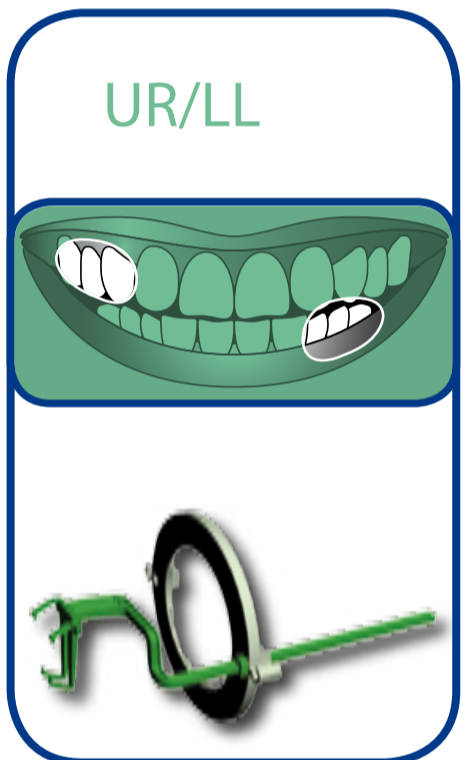


Endo

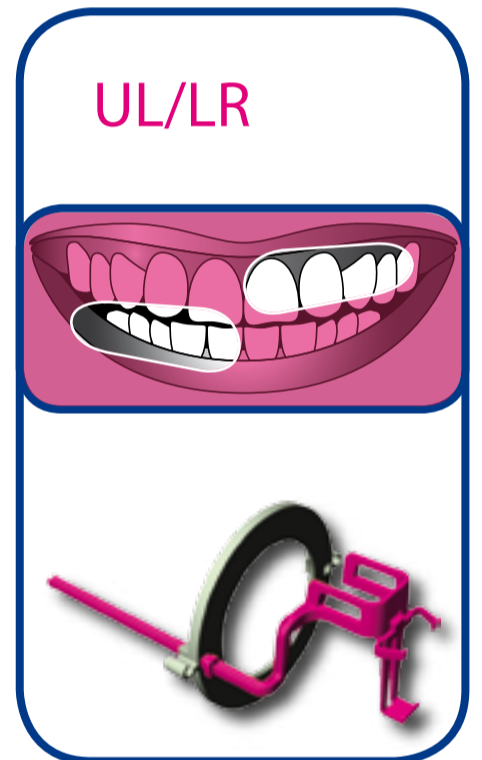

Assembling for all horizontal positions (Bitewing and Posterior)

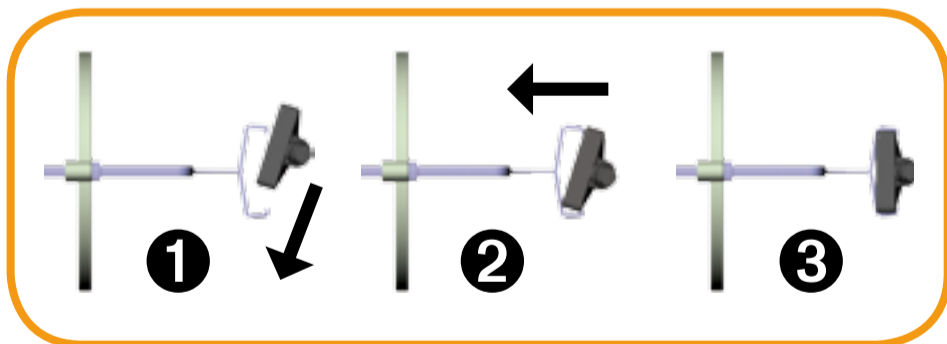

Assembling for all vertical positions (Periapical and Endo)

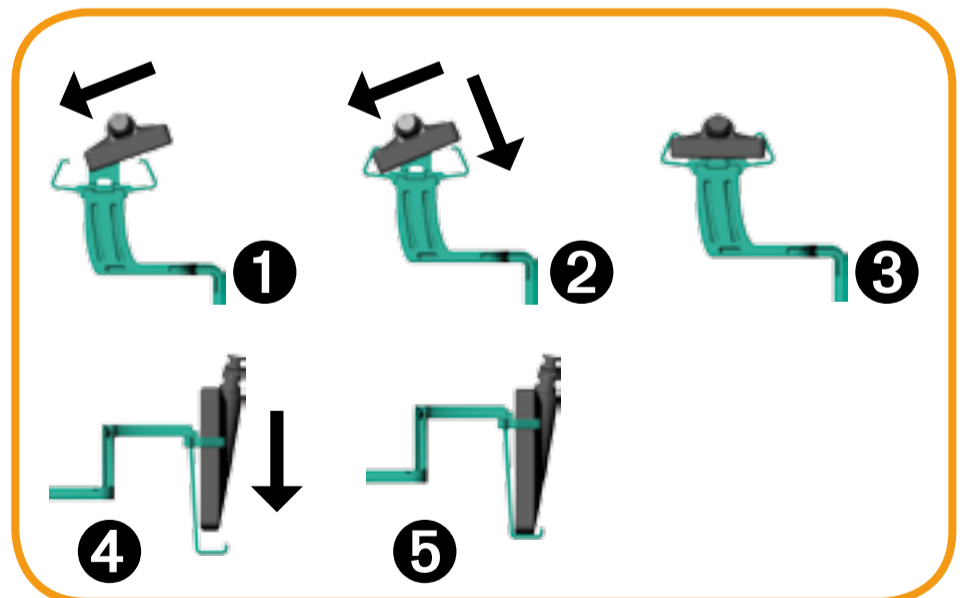

Positioning ring
Rotate sensor holder to change position and use the other fixation point of positioning ring.

General hygiene notes
Responsible product; must be sterilized in autoclave or thermodisinfectant before use!
First place sensor in sensor sleeve before placing in the Owandy sensor holder.
Owandy sensor holder must be sterilized in autoclave or thermodisinfectant after use on every patient.